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EN - Chinese Rubber Cupping Therapy Cups

Usage Instructions: The cupping procedure should be performed in a room with a temperature of 20-22°C and should last up to 15 minutes. Depending on the treated area, a lying position on the stomach, side, or sitting position is recommended. The cups should be placed symmetrically on both sides of the body. Wash the skin with soap and water, then apply oil. Shave areas with dense hair. Hold the lower edge of the cup with both hands while pressing the top with your thumb to expel air. Gently press the cup against the selected point on the body and release your fingers. Cover the patient with a blanket and leave for up to 15 minutes. Afterward, press the skin with your thumb to release the air and remove the cup. Reapply oil to the skin. During the procedure, a sensation of warmth and stretching should be felt, but not pain. After the procedure, the person should rest in bed.

Warnings: Consult a doctor before use. For the first session, use smaller cups. After each use, clean thoroughly with a disinfectant or an alcohol-soaked cotton pad. Do not use bleach, strong detergents, or rough surfaces for cleaning. Do not wash in a dishwasher.

Avoid high temperatures for disinfection. Do not boil or steam the cups. Exercise caution to prevent excessive suction pressure. Do not use on children or individuals over 80 years old. If pain occurs during the procedure, contact a doctor immediately. If any concerning symptoms appear (nausea, excessive drowsiness, palpitations, etc.), discontinue use immediately. This product is not a toy. Use only as intended. Keep out of reach of children.

Contraindications: Do not use in cases of skin damage or inflammatory conditions (allergic, fungal, bacterial, purulent); fragile blood vessels; blood clotting disorders; shortness of breath; autoimmune diseases; multiple sclerosis; anemia; general weakness; fever; unstable blood pressure; pregnancy; menstruation; subcutaneous hemorrhages and swelling; cancer; tuberculosis; seizures; after alcohol consumption. Do not use on individuals with heart disease or those taking anticoagulants. Do not use immediately after a meal.

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