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EN - Bunion Separator, 2 pieces; Material: SEBS

Instructions for use: Wash and dry your feet before the first use. Gently insert your toes into the appropriate compartments. The separator should feel comfortable and should not exert excessive pressure on your feet. Start with short periods of use to allow your feet to adjust, gradually increasing the wearing time. Wear shoes that are not too tight in the front area. After removing the separators, wash them thoroughly with soap and water and let them dry.

Warnings: Read the instructions before use and keep them for future reference. Avoid using on cracked, irritated, or damaged skin. If you feel pain, irritation, or discomfort, discontinue use immediately. It is not recommended to use during sleep or for prolonged periods without a break. Consult a doctor before use if you have foot conditions. Monitor your feet and body's reactions while using the separators. If you notice any concerning symptoms, stop using the product and consult a doctor. Regularly check the condition of the separators; if you notice any damage or excessive wear, replace them with new ones. This product is not intended to replace professional medical care. The product is not a toy. Keep out of reach of children.

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