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EN - Gel Metatarsal Band

Instructions for Use: Put the band directly on your foot, under your sock and shoe. You may sprinkle your foot with talcum powder or baby powder to prevent irritation. Make sure your foot is clean and dry before putting on the band. Check that the band fits properly and does not cause pressure or discomfort.

Warnings: Read the instructions before use and keep them for future reference. Use the product as intended. It is recommended to consult a doctor before use to confirm that there are no contraindications for wearing the product. Apply the product to a clean and dry foot. To extend its lifespan, always clean it with lukewarm water and, after drying, sprinkle it with a small amount of talcum powder. Clean the product regularly. Remember that regularly cleaning sweat from under the insert and your feet will ensure comfort and long-term use. In case of serious foot problems, consult a doctor. Avoid contact with sharp objects to prevent damage to the band. Store the band away from direct heat sources and sunlight. The band is designed for multiple use. However, if you notice any signs of wear, damage, or loss of elasticity, stop using it and replace it with a new one. If any irregularities occur while using the band, stop using it immediately and consult a doctor.

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