PL - Stabilizator kolana

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EN - Knee Stabilizer

How to Use: Place the stabilizer with the opening positioned over the knee. Then wrap the Velcro straps around the leg, ensuring they fit snugly to the anatomical curves, and fasten the Velcro. Adjust all straps to the proper position, ensuring they are not too tight to avoid discomfort. If possible, ensure the area supported by the stabilizer is adequately elevated and relieved to minimize the risk of further injury. If redness, irritation, or discomfort occurs, remove the stabilizer immediately. If knee pain persists, consult a doctor for proper diagnosis and further action.

Warnings: Follow the instructions provided with the product to ensure proper functionality of the orthosis. Incorrect adjustment or improper fastening of the straps may impair functionality and reduce comfort. Before each use, check the condition of the device. Do not use it if the device is damaged. Do not use the orthosis while driving. The orthosis is intended for use by a single patient and should not be used by another person. The conditions and duration of use should be determined by a doctor or physiotherapist. Avoid contact with substances containing oils (ointments, creams, lotions). If increased pain, swelling, skin irritation, or any other adverse reaction occurs while using the orthosis, contact your doctor immediately.

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