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Stabilizator barku.

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MATERIAŁ: 75% Neopren, 25% Poliester.

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Shoulder Stabilizer.

USAGE INSTRUCTIONS: Thread the strap through the closure loop and secure the hook-and-loop fastener before placing it on the shoulder. Then wrap the cross strap under the opposite shoulder, fastening it in the center of the chest. Adjust all straps to the desired position. Avoid overtightening to prevent discomfort. Next, adjust the compression pad on the shoulder, ensuring it is precisely aligned with the anatomical curves. If possible, make sure the area supported by the stabilizer is properly elevated and relieved to minimize the risk of further injury. It is recommended to check the shoulder every 15 minutes. If redness, irritation, or discomfort occurs, immediately remove the stabilizer. If shoulder pain persists, obtaining a proper diagnosis from a

doctor is essential to determine the next steps.

WARNING: Do not use the stabilizer during sleep, near heat sources, or while exercising. Do not apply additional pressure to the stabilizer. Ensure proper air circulation around the stabilizer. If skin discoloration occurs, discontinue use immediately. Not suitable for children, elderly individuals, or those unable to operate the stabilizer independently. Do not use on irritated or damaged skin. Do not use the product in conjunction with creams applied to the skin or in cases of rheumatoid arthritis, poor circulation, nerve damage, sensory disorders, skin diseases, vascular disorders (Raynaud's disease), diabetes, or pregnancy. For external use only. Do not use the stabilizer if allergic to latex. The product contains natural latex rubber, which may cause allergic skin reactions.

IMPORTANT: To facilitate blood flow, the stabilizer should be removed for at least 10 minutes three times a day. Plastic packaging can be dangerous. To avoid the risk of suffocation, keep it out of reach of children and infants.

WASHING INSTRUCTIONS: Hand wash only. Do not bleach. Line dry. Do not iron. Do not dry clean.

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